

User manual STD combination-test

You will receive a test-set including a tube. In the end you collect all swab sticks and/or urine in the same tube.

The chlamydia and chlamydia/gonorrhea combination test for women contains a Hologid APTIMA kit including:


- Three white cotton swabs (for vagina, anus and potentially throat)
- A white tube without a small window, which is filled with transport liquid
- An application form
- A blue transport envelope with RigidSafetyBag
- A user manual for collection and transport

The Chlamydia and chlamydia/gonorrhea combination test for men contains a Hologid APTIMA kit including:

- Two white cotton swabs (for anus and potentially throat)
- A urine pipet (for penis/urine)
- A yellow tube with a small window, which is filled with transport liquid
- An application form
- A blue transport envelope with RigidSafetyBag
- A user manual for collection and transport

User manual culture/swab collection

- Mouth: Insert one cotton swab reaching the tonsils in the back of the mouth to collect mucus.
- Vagina: Insert the cotton swab into the vagina, and rotate the cotton swab for a few seconds. Carefully put the tip in the tube, avoiding external surfaces, and break it.
- Penis: Catch some urine and use the pipet to collect a small proportion. Proceed by putting it in the tube until the window is filled halfway through.
- Anus: Insert one cotton swab rectally and remove it. Carefully put the tip in the tube, avoiding external surfaces, and break it.

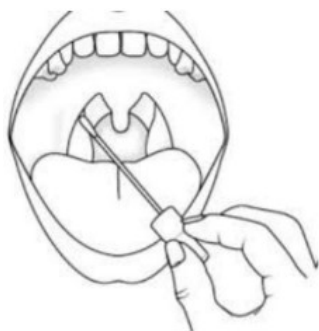

Recently a QR-code sticker was included on the tube and the application form. Close the tube including the 2-3 sticks and send the package via post to the laboratory. **IMPORTANT: SEND THE APPLICATION FORM TOGETHER WITH THE TEST-KIT TO THE LAB!** After 3-4 business days go to soastudentarts.nl. Fill in your client number at “results” and you will receive your results anonymously.

Contact us

E-mail: info@soastudentarts.nl
Phone: +31 (0)50 211 0 911