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Tel: +31 (0) 50 211 0 911

*For English version see other side

User manual STD combination-test (women)

You will receive a test-set including a tube. You can choose which body areas (vagina, anus, mouth) you want to include in the test. **Make sure that all used cotton swabs are collected in this tube.**

Please note: the testing material has an expiration date. Check the expiration date on the test tube and on the packaging of the cotton swabs.

The chlamydia and chlamydia/gonorrhoea combination test for women contains a Hologic Aptima kit, including:

- A test tube filled with transport liquid
- Three cotton swabs (for vagina, anus and mouth)
- An application form
- A blue or white transport envelope with RigidSafetybag
- A user manual for collection and transport

Collection instructions:

- Vagina: insert the cotton swab into the vagina, and rotate the cotton swab for a few seconds. Carefully put the cotton swab in the tube, avoiding external surfaces, and break it.
- Anus: insert one cotton swab rectally and remove it. Carefully put the cotton swab in the tube, avoiding external surfaces, and break it.
- Mouth: insert one cotton swab reaching the tonsils in the back of the mouth to collect mucus. Carefully put the cotton swab in the tube, avoiding external surfaces, and break it.

- Close the tube including the cotton swabs, and place it according to the instructions in the RigidSafetyBag.
- Put the RigidSafetyBag in the return envelope.
- **IMPORTANT: ALSO INCLUDE THE APPLICATION FORM IN THE RETURN ENVELOPE!**
- Send the package by post to the laboratory using the return envelope.
- After 3-4 business days go to SoaStudentArts.nl. Fill in your client number at 'results' and you will receive your results anonymously.

Contact

E-mail: info@soastudentarts.nl

Phone: +31 (0)50 211 0 911

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