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Tel: +31 (0) 50 211 0 911

*For English version see other side

User manual STD combination-test (men)

You will receive a test-set including a tube with a yellow sticker. If you choose to use the cotton swabs (to test for anal and/or throat infections), make sure that **all used cotton swabs and urine are collected in the same tube with the yellow sticker**.

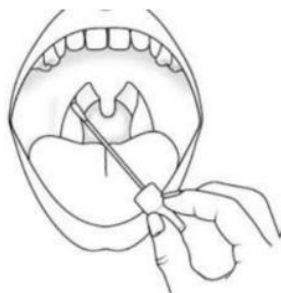
Most of the material has an expiration date. Check the expiration date on the test tube and on the packaging of the cotton swabs.

The chlamydia and chlamydia/gonorrhea combination test for men contains a Hologic Aptima kit, including:

- A tube with a yellow sticker with a small window, which is filled with transport liquid
- A pipet (for transferring urine)
- Two white cotton swabs (for anus and throat, in case you want to test these areas as well)
- An application form (**please note: make sure to also send this form to the lab!**)
- A blue or white transport envelope with RigidSafetybag
- A user manual for collection and transport

Collection instructions:

- In case you will make use of the cotton swabs: **start** with the swabs and **end** with the urine collection, so that the swabs are already in the tube before adding the urine.
- Anus (optional): insert a cotton swab rectally 3-5 cm deep and turn it around for 5-10 seconds. Carefully put the cotton swab in the tube with the yellow sticker, avoiding external surfaces, and break it.
- Throat (optional): insert a cotton swab deep into your mouth and swipe the sides and back of your throat to collect mucus. Carefully put the cotton swab in the tube with the yellow sticker, avoiding external surfaces, and break it.
- Penis: collect some urine by urinating in a **clean cup**, and use the pipet to collect a small proportion. Proceed by putting it in the tube with the yellow sticker until the window is filled halfway through.

- Close the tube including the urine and (potentially) the cotton swabs, and place it according to the instructions in the RigidSafetyBag.
- Put the RigidSafetyBag in the return envelope.
- **IMPORTANT: ALSO INCLUDE THE APPLICATION FORM IN THE RETURN ENVELOPE!**
- Send the package by post to the laboratory using the return envelope.
- After 3-4 business days go to [SoaStudentArts.nl](https://www.soastudentarts.nl). Fill in your client number at 'results' and you will receive your results anonymously.

Contact us

E-mail: info@soastudentarts.nl

Phone: +31 (0)50 211 0 911

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