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\*For English see other side

## Instructions STD test women (vaginal)

These instructions are meant for our chlamydia and gonorrhea tests (vaginal). Check before you test if the kit is complete. Also check the expiration date on the test tube and the packaging of the cotton swabs.

The test kit for women (vaginal) contains:

- A test tube filled with transport liquid
- A cotton swab
- An application form
- A white (plastic) shipping envelope
- A Rigid Safety Bag
- A tube holder

Instructions:

1. Get the application form and write at 'afnamedatum' which date it is today (the date you're carrying out the test).
2. Remove the cap of the test tube by rotating it, and place the tube in the tube holder. Make sure that the liquid stays inside the tube.
3. Get the cotton swab by holding the black line on the cotton swab between your thumb and index finger. Avoid contact with other surfaces.
4. Insert the cotton swab 5 cm into your vagina, and swipe along the vaginal wall for 30 seconds.
5. Place the cotton swab into the test tube, avoiding external surfaces, and break it at the black line. Close the tube by firmly rotating the cap onto the tube.

6. Place the test tube according to the instructions in the Rigid Safety Bag. Remove the white strip on the Rigid Safety Bag to seal the bag.
7. Put the Rigid Safety Bag in the white (plastic) shipping envelope.
8. **Please note: also include the application form in the shipping envelope!**
9. Send the shipping envelope (including the Rigid Safety Bag with the test tube, and the application form) to the lab by placing it in an orange PostNL postbox before 5 pm on a weekday. You don't have to put stamps on it.
  - Is the outside temperature below 0 or above 30 degrees celsius? In that case, bring the package to a PostNL postal office.
10. After 3-4 business days go to the results page on [SoaStudentArts.nl](https://www.soastudentarts.nl). Fill in your client number and you will receive your results anonymously.

### Contact

E-mail: [info@soastudentarts.nl](mailto:info@soastudentarts.nl)

Tel: +31 (0) 50 211 0 911

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